



Chip Development

QUICK START GUIDE FOR NURSE CALL RECEIVER (NCR)



<p>Programme/Enroll New Transmitter</p>	<ul style="list-style-type: none"> • Step 1: Insert 2 x AA Alkaline batteries into NCR. • Step 2: Make sure the Transmitter is not connected to sensor pad/mat and out of sleep mode. To do that, press and hold the button located in the middle of the transmitter for up to 10 seconds until both LEDs flash. • Step 3: Bring the transmitter close to NCR. Flick programming switch at NCR from W to P. Channel 1 starts flashing. • Step 4: Use the “Reset Button” at the front bottom of NCR to change and select the channel you want to use. • Step 5: Press and hold the button located in the middle of the transmitter for up to 10 seconds until you hear a beep from NCR and the flashing LED has moved to the next channel. When you do not hear the beep, release the button at Transmitter and press and hold it again for over 10 seconds. • Step 6: Switch the programming switch back to W. • Step 7: Plug transmitter into Pad/Floor Sensor Mats for testing. In the case of Wireless Call Pendant, just press the call button to test.
<p>Delete All Transmitters</p>	<p>To clear the memory of NCR and delete all existing transmitters programmed, please do the following steps:</p> <p>Step 1: Open the back cover and remove at least one battery from NCR.</p> <p>Step 2: Press and hold “Reset Button” and insert the battery back to NCR while holding “Reset Button”.</p> <p>Step 3: Keep holding the “Reset Button” for up to 15 seconds until LED starts flashing.</p> <p>Now the memory of NCR has cleared and you need to add all transmitters again. (See above)</p>